



Contour® Optics by SOLA

DISPENSING GUIDE

1. Confirm availability.

Confirm that Contour® Optics eyewear is available in the patient's prescription.


- +1.00 to -5.00 spheres, out to a -1.00 cyl
- 5.25 to -6.00 spheres, no cyl
- 0.25 diopter steps

2. Measure monocular PDs.

Use a pupillometer to take monocular distance PDs.

3. Ensure a good frame fit.

- Ensure that pupils will be in the correct location when the completed frame is adjusted. Pupils should always be located slightly above midline, about the same height as the temples.

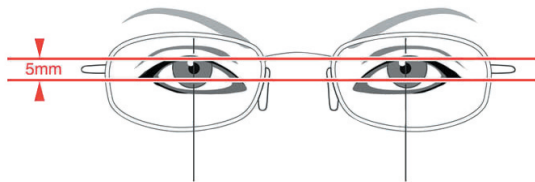

- As with all frames, check to see that the nose pads are appropriate for the patient's bridge width. (Note that an additional set of wider nose pads will be provided with finished eyewear to accommodate narrower bridges.)

4. Confirm PD and frame match.

Make sure the frame selected by the patient is available for the patient's PD. If not, suggest an alternate frame size.

Refer to the Contour Optics Order Form, available from SOLA Technologies, for details about frame sizes, shapes and color options.

5. Call SOLA Technologies to order.

SOLA Technologies
866-289-7652 (toll-free)



